NTI-Kahla GmbH • Rotary Dental Instruments
Im Camisch 3 • D-07768 Kahla/Germany
Tel. +49(0)36 424-573-0 • Fax +49(0)36 424-573-29
e-mail: nti@nti.de • www.nti.de

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Instructions for use for rotary dental and surgical instruments

Intended use and correct application

Rotary dental and surgical instruments are intended for dentistry and must only be used by dentists and other qualified professionals who are familiar with use of these instruments based on their training and experience.

Indication:

The symbols (EN ISO 21531) next to the figure number (see Dental Surgery catalogue www.nti.de, and on the packaging) denote the possible applications in the dental practice.

Contraindication:

The instruments must not be used for methods other than those specified by the symbols. The speeds given must not be exceeded. Improper use leads to increased risk and inferior results.

The shanks of the instruments are manufactured in accordance with the Standard ISO 1797 and should only be used in combination with a drive for rotary instruments, which complies with ISO 14457. The operator should ensure at the beginning of treatment that the rotary instrument is correctly secured in the drive. Strict observance of the safety recommendations achieves optimum working conditions and long service life of the dental and surgical instruments.

Improper use results in premature wear, destruction of the instruments and drive, injury to tissue and risk to the operator, patient and third party.

Correct use:

- Adhere to the speed indicated on the packaging!
- Excessive speeds near the tooth pulp risk the vitality. The same applies for an insufficient cooling technique.
- Excessive speeds with oversized working parts endanger the stability of the instruments.
- The drive systems, conforming to ISO 14457, must be in perfect technical condition. Ensure adequate water cooling (min. 50 ml/min.) on the working section with speeds greater than 1,500 rpm.
- Additional external cooling is required in the case of instruments with an overall length of >19 mm and a head diameter of >1.8 mm (ISO -018).
- Do not impair wetting of the areas to be prepared by using an incorrect suction technique (use with intraoral suction). Deflection of the spray may result in thermal damage.
- Operate using a contact pressure of 0.3 N to a maximum of 2 N depending on the type of preparation.
- Avoid blocking due to excessive contact pressure, tilting and leverage (increased risk of fracture, heat build-up, damage of fillings).
- Insert the instruments as far as possible into the chuck and check they are firmly seated.
- Extremely sharp instruments wear more quickly due to their small size in the front section. Prepare particularly sensitively using minimal contact pressure with these instruments to prevent shortening the service life.
- Apply Turbo instruments over the entire surface.
- Any surface roughness produced with a diamond coating of >ISO 524 must be removed.
- Damaged, bent or no longer true-running instruments should be separated immediately and should no longer be used.
- Use of protective glasses and face mask is recommended, depending on the type of application.
- Only use diamond discs with a disc guard.

Preparation instructions:

- Rotary dental and surgical instruments are supplied mechanically clean. They are non-sterile.
- Rotary instruments must be placed in a cleaning solution, cleaned, disinfected, dried, checked and sterilised before first use on a patient and immediately after each use.
- Only disinfectants that are suitable and approved for rotary instrument may be used.
- Use of unsuitable disinfectant and/or excessively long immersion in the disinfectant can result in corrosion (increased fracture risk) and discoloration as well as removal of the colour ring.
- Alkaline drill solutions must not be used for disinfecting polishers.
- Never use alcohol or disinfectants containing alcohol for rotary instruments, in particular polishers!
- Colour-anodised aluminium components (e.g. Retopin mandrels) lose their colour during use of standard cleaning processes and in cleaning and disinfection appliances. Special coordinated cleaning and disinfectant solutions should be used during preparation.
- Steel instruments (ISO 310) are unsuitable both for mechanical processing and steam steriliser. Steel instruments should be changed for the respective tungsten carbide instruments.
- Use of an instrument stand is recommended to avoid contact damage.
- Instruments must be sterilised in an autoclave (Class B) at 134°C (270°F) / 5 min. (validated process). Only use fully desalinated water.
- Observe "Hygiene requirements when preparing medical products". (Recommendations of the Commission for Hospital Hygiene and Infection Prevention (KRINKO) at the Robert-

Koch Institute (RKI) and the German Federal Institute for Drugs and Medical Devices (BfArM) or the current legal and hygiene regulations in your country!

- Detailed preparation recommendations according to DIN EN ISO 17664 ► www.nti.de

Reusability:

Single-use products (identified by  on the packaging) are not approved for reuse.

The safety of these products cannot be guaranteed with reuse, as there is the risk of infection and/or the product is no longer safe.

Frequent re-preparation does not affect or restrict use of the instruments, as the end of the product service life is determined by wear and damage during use.

Storage and care instructions:

Store dry at room temperature approx. 20° C and 40-60 % humidity, protected against recontamination.

Instruments should always be protected against dust, chemicals, acids, heat and extreme temperature fluctuations.

Transport and disposal:

After cleaning and disinfecting, dispose of instruments according to national or local regulations.

Further instructions for use, e.g. regarding surgical or endodontic instruments are available at ► www.nti.de

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